





AGENDA

- 1. Gloves practice guidelines and recommendations
- 2. Surgical gloves portfolio Hands on training
- 3. Examination gloves portfolio Hands on training



WHO recommendation for gloves indication

STERILE GLOVES INDICATED

for any surgical procedure

EXAMINATION GLOVES INDICATED

during direct and indirect patient exposure
when potential for touching blood,
body fluids, secretions, excretions and
items visibly soiled by body fluids

GLOVES NOT INDICATED (except for CONTACT precaution)

during direct and indirect patient exposure
when there is no potential for exposure to blood or body fluids or
contaminated environment



WHO consensus recommendation on the use of gloves

Use of gloves does not replace hand hygiene by hand rubbing or washing



- Wear gloves when contact with blood or other potentially infectious materials, mucous membranes, or non-intact skin will occur
- Remove gloves after caring for a patient.
- Do not wear the same pair of gloves for the care of more than one patient.
- When wearing gloves, change or remove gloves during patient care if moving from a contaminated body site to another body site within the same patient or when touching the environment.
- Gloves are single-use products

Change gloves



General recommendations on the use of gloves

- Right size and close fit is important for safe grip, sensitivity and working comfort
- Skin lesions shall be covered with a waterproof plaster while wearing gloves
- Hands should be dry and clean when donning the gloves
- When changing gloves, sweat shall be allowed to evaporate or dried
- Oil-based moisturizers can damage the protective functions of the gloves. Generally, the use of moisturizers under occlusive gloves is not recommended.
- Appropriate hand care after the use of gloves is essential for healthy skin







B. Braun Examination and Protective Gloves




DALIN BIBRAUN



Hands-on training

Donning of non-sterile gloves


How to don non-sterile gloves (I):

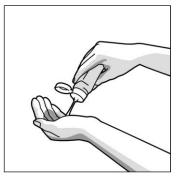


1. When the hand hygiene indication occurs before a contact requiring glove use, perform hand hygiene by rubbing with an alcohol-based handrub or by washing with soap and water.



2. Take out a glove from its original box.

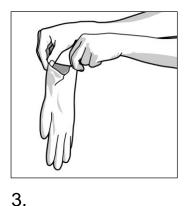

Touch only a restricted surface of the glove corresponding to the wrist (at the top edge of the cuff).

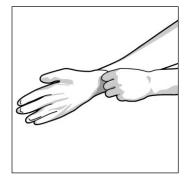

4. Don the first glove.




Donning of non-sterile gloves

How to don non-sterile gloves (II):




5.

Take the second glove with the bare hand and touch only a restricted surface of glove corresponding to the wrist.

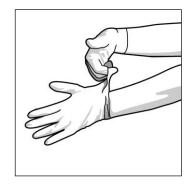

6.

To avoid touching the skin of the forearm with the gloved hand, turn the external surface of the glove to be donned on the folded fingers of the gloved hand, thus permitting to glove the second hand.

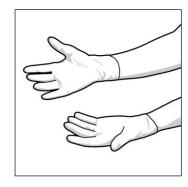

7.

Once gloved, hands should not touch anything else that is not defined by indications and conditions for glove use.



Removing of non-sterile gloves

How to remove non-sterile gloves:




1.

Pinch one glove at the wrist level to remove it, without touching the skin of the forearm, and peel away from the hand, thus allowing the glove to turn inside out.




2.

Hold the removed glove in gloved hand and slide the fingers of the ungloved hand inside between the glove and the wrist. Remove the second glove by rolling it down the hand and fold into first glove.

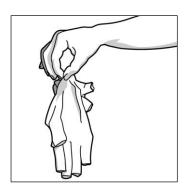

3.

Discard the removed gloves.

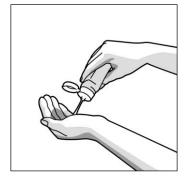

4.

Perform hand hygiene by rubbing with an alcoholbased handrub or by washing with soap and water.



Vasco® Nitril white

POSITIONING: high quality Nitrile glove with white color





Vasco® Nitril white



SIZE	SALES UNIT	REF	SALES UNIT	REF
XS	100	9207902	150	9208402
S	100	9207910	150	9208410
M	100	9207929	150	9208429
L	100	9207937	150	9208437
XL	90	9207945	135	9208445

- powder-free examination gloves according MDR (EU) 2017/745, EN 455
- protective gloves according PPE Regulation EU 2016/425 (Cat. III), EN 420, EN 374 (Type B)
- online chlorinated
- typical average wall thickness at palm: 0.06 mm



Vasco® Nitril blue

POSITIONING: high quality Nitrile glove with blue color





Vasco® Nitril blue



SIZE	SALES UNIT	REF
XS	150	9209809
S	150	9209817
M	150	9209825
L	150	9209833
XL	135	9209841

- powder-free examination gloves according MDR (EU) 2017/745, EN 455
- protective gloves
 according PPE
 Regulation EU
 2016/425 (Cat. III), EN
 420,
 EN 374 (Type B)
- online chlorinated
- typical average wall thickness at palm: 0.06 mm



Vasco® Nitril Soft white

POSITIONING: Box of 200 pieces for hospitals





Vasco® Nitril Soft white



SIZE	SALES UNIT	REF
XS	200	9201100
S	200	9201110
M	200	9201120
L	200	9201130
XL	180	9201140

- powder-free examination gloves according MDR (EU) 2017/745, EN 455
- protective gloves according PPE Regulation EU 2016/425 (Cat. III), EN 420, EN 374 (Type B)
- online chlorinated
- typical average wall thickness at palm: 0.06 mm



Vasco® Nitril Soft blue

POSITIONING: Box of 200 pieces for hospitals





Vasco® Nitril Soft blue



SIZE	SALES UNIT	REF
XS	200	9201107
S	200	9201115
М	200	9201123
L	200	9201131
XL	180	9201149

- powder-free examination gloves according MDR (EU) 2017/745, EN 455
- protective gloves according PPE Regulation EU 2016/425 (Cat. III), EN 420, EN 374 (Type B)
- online chlorinated
- typical average wall thickness at palm: 0.06 mm



Vasco® Nitril Ocean-Blue

POSITIONING: Box of 150 pieces, high quality with competitive pricing





Vasco® Nitril ocean-blue



SIZE	SALES UNIT	REF
XS	150	8208001
S	150	8208002
M	150	8208004
L	150	8208005
XL	135	8208006

- powder-free examination gloves according MDR (EU) 2017/745, EN 455
- protective gloves
 according PPE
 Regulation EU
 2016/425 (Cat. III),
 EN 420,EN 374
 (Type B)
- latex-free inner coating
- typical average wall thickness at palm: 0.06 mm



Tactile properties of Vasco® Nitril gloves

textured structure in finger area for safe grip & instrument handling

excellent sensitivity due to light glove design





Vasco[®] Nitril family



- Nitrile gloves are easy to don because of the material characteristics
- Rolled rim provides cuff strength for further ease in donning
- Good elasticity of the material ensures a comfortable fit of the gloves



Vasco® Vinyl Powder-free

POSITIONING: Box of 100 pieces, with very competitive pricing





Vasco® Vinyl Powder-Free



SIZE	SALES UNIT	REF
XS	100	9209919
S	100	9209927
M	100	9209939
L	100	9209942
XL	90	9209953

- powder-free examination gloves according MDR (EU) 2017/745, EN 455
- protective gloves according PPE Regulation EU 2016/425 (Cat. I), EN 420, EN 374 (Type C)
- with latex-free inner coating
- typical average wall thickness at palm: 0.065 mm
- Free of DOP/DEHP (dioctyl phthalate/diethylhexyl phthalate)




Summary examination gloves positioning

Product	Min. wall thickness (palm)	AQL	Surface	Price
Vasco® Sensitive	0,12 mm	1,5	chlorinated/coated	High price
Vasco® Basic	0,10 mm	1,5	chlorinated	Medium price
Vasco® Nitril Soft white	0,05 mm	1,0	chlorinated	Lowest price
Vasco® Nitril Soft blue	0,05 mm	1,5	chlorinated	Lowest price
Vasco® Nitril light	0,06 mm	1,5	coated	Medium price
Vasco® Nitril white / blue	0,05 mm	1,0	chlorinated	Medium price
Vasco® Nitril semilong white	0,05 mm	1,0	chlorinated	Lowest price
Vasco® Nitril long	0,08 mm	1,0	chlorinated	High price
Vasco® Vinyl PF	0,065 mm	1,5	n/a	Low price



Colour concept examination gloves packages

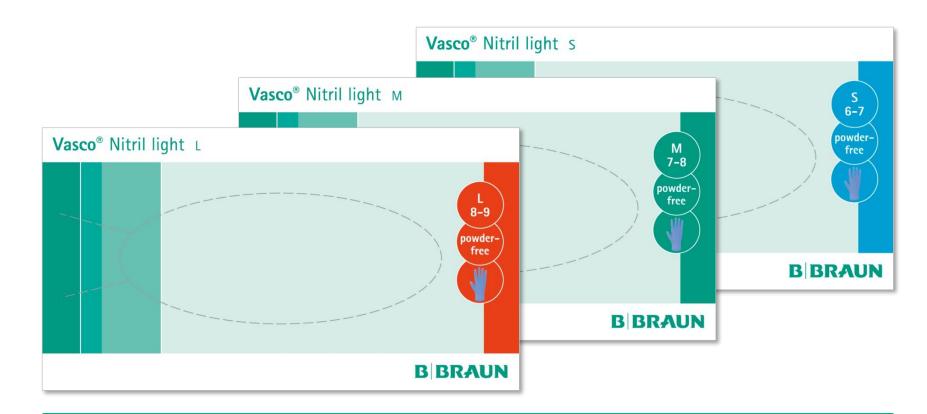

For Examination Gloves each size has a different color, no matter which type of glove.



Upselling of B. Braun examination gloves – accessories

- wall bracket for glove dispenser-boxes
- stainless steel
- available options:
 - Single (package)
 - TRIO (3 packages)
 - QUATRO (4 packages)

